## Keyboard

Arrangement of keyboard components is shown in figure 5.6.

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### START key

Pressing START key after:

- selection of treatment program or user-defined program in program mode,
- edition of parameters in manual mode

causes the start of the treatment procedure.

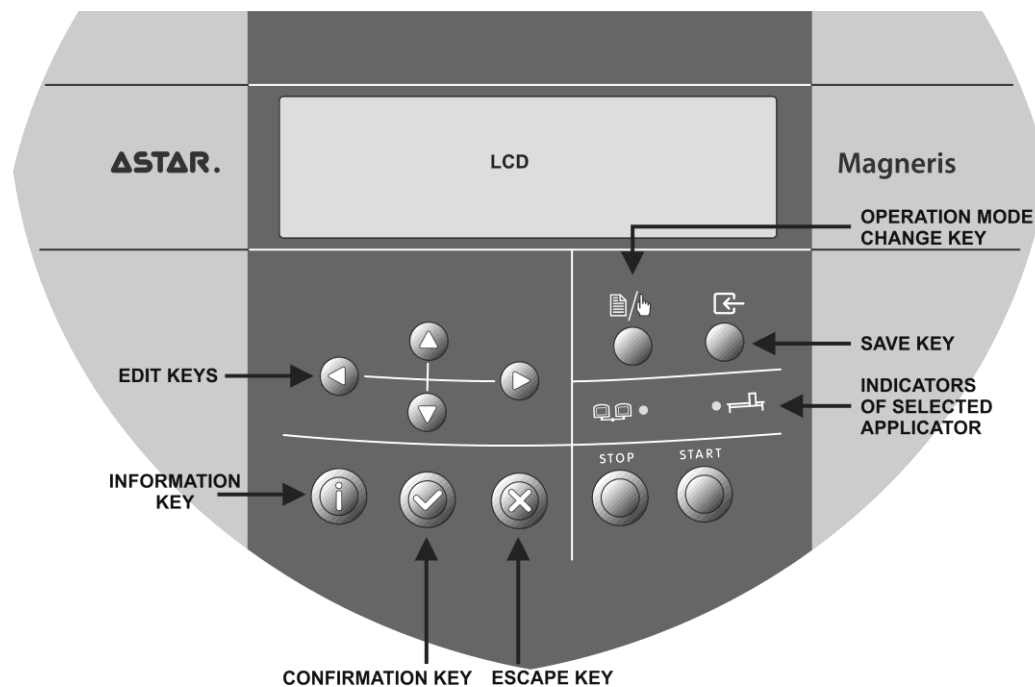
Pressing START key after interruption of a treatment procedure (pause) makes it possible to continue the procedure.

### STOP key

Pressing STOP key while performing treatment procedure causes the interruption of procedure. The unit enters the waiting state (pause). The timer will stop then.

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## 7.1 Operation with preset treatment programs

The simplest method to use the unit is to use its preset programs or treatment sequences. The unit includes a base of several dozens of most frequently met disorders together with suggested treatment types and parameters. The list of all diseases with parameters is included in Appendix A at the end of the manual.

**The values of the preset treatment programs parameters are based on the available literature data and they are determined as average values. Parameters should be treated exclusively as indications. Sole responsibility for application of preset treatment programs rests with the user.**

### Schematic procedure for treatments:

Step	Description
1.	Switch on the unit. Depending on the <b>Start-up menu</b> configuration (see section 6.2.6) the unit starts up either in the <b>PROGRAM GROUPS</b> menu mode or in a set treatment program (or user program) that was used during the previous treatment procedure. In the latter case, in order to select a different program from the <b>PROGRAM GROUPS</b> menu, press the <b>X</b> key. When no program change is necessary, proceed to step 5.
2.	If the unit is in manual mode, press the key
3.	Press the   keys to choose the <b>Preset programs</b> option, and press the  key to confirm.
4.	Using the   keys, choose the position from the list.
5.	Prepare the patient for the treatment. Set up the applicator that will be used during the treatment procedure.
6.	Press the <b>START</b> .
7.	Press the   keys to adjust the magnetic-field induction, if required.

Press the **i** key to display an information screen containing parameters of the relevant treatment program. Definitions of symbols are provided in section 5.5.

In order to close the information window (showing the name or parameters), press the **i** or **X** key.

**NOTE:** If two magnetic field applicators are connected, the applicator selection window displays when the **START** key is pressed. Press the buttons to select an applicator. When the applicator type is selected, press the **START** key again. The unit starts generating magnetic field. An indicator denoting the selected applicator lights up on the keyboard.

In order to pause the treatment procedure, press the **STOP** key. The time counter will stop. In order to continue the treatment procedure, press the **START** key. If you wish to return to the **Preset programs** menu, press the **X** key.

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